

How to administer



1 Preparation

Administer a **topical anaesthetic agent** into the conjunctival sac, following the product manufacturer's instructions.



Ensure the canaliculus is open and suitable for Lacrifill administration by performing **punctal irrigation**, confirming good lacrimal drainage.

2 Set-Up

Remove the syringe from the packaging and **remove the cap** from the end of the syringe.



Fit the accompanying cannula onto the end of the syringe, ensuring it is secured as **tightly as possible**.



With the syringe **facing upwards**, it is vital to **prime** the cannula by gently applying steady pressure until approximately **0.1 mL of gel** is expressed through the tip.



3 Administration

Lacrifill® can be administered at the **slit lamp** or inserted using **loupes** as the patient reclines in a chair.



Gently **pull down the lower eyelid** to expose the punctum and hold the eyelid firmly.

Carefully insert the cannula into the **inferior punctum**.

Apply **steady, gentle pressure** to the syringe, filling the canaliculus corresponding approximately **0.2 mL of gel**.

After completing the administration in the first eye:

Re-secure the cannula, then **re-prime** it by expressing an additional 0.1 mL of gel through the tip.

Repeat the administration process for the second eye.

The administration of Lacrifill® relies on 3 key steps to ensure a successful procedure and a comfortable experience for patients:




1 Secure cannula installation

Ensure the cannula is firmly tightened onto the syringe.



2 Upward priming

Gently express 0.1 mL upward to prime before administration.

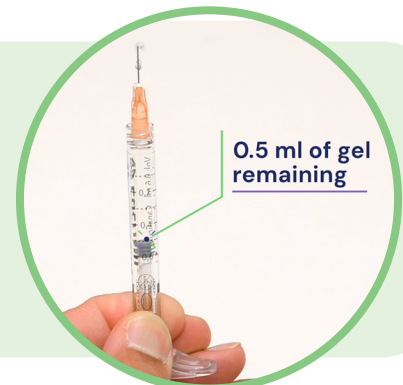

3 Gentle, consistent pressure

Use smooth, even pressure during administration to ensure complete filling.



Scan this QR code to watch the Lacrifill administration video



Scan this QR code for full Lacrifill instructions for use